

# “Alles gurgelt!”

## Gargle with Us!



1



Get your access barcode by visiting [allesgurgelt.at](https://allesgurgelt.at) and receive up to **4 test sets** by showing the barcode at any BIPA outlet in Vienna.

2



Enter [lead-horizon.org/public](https://lead-horizon.org/public) to access the WebApp and register by inputting the sample number. After registering, you will receive a **personalised barcode** that enables you to obtain new test sets in the future. Follow the instructions in the WebApp and perform the test.

3



**Confirm your identity** for the test by holding your passport, ID card or e-card towards the camera.

4



If you agree to a **video recording** of the test procedure, a **medical lab report** will be issued for you for going to a hair salon or other shops/service providers, if your test is negative. If you skip this step, you will receive a test result, but no officially valid medical report.

5



Now perform the **test**. You can gargle or rinse your mouth with the saline solution.

6



After performing the test, put the **sample** back into the LEAD Horizon packaging.

7



Now just hand in the test at **any REWE outlet in Vienna** (BILLA, BILLA Plus, BIPA and Penny as well as all petrol station shops of BP-Merkur Inside, JET-BILLA Stop & Shop, SHELL-BILLA Unterwegs).

8



If you **hand in your sample before 9 a.m.**, you will receive your test result via email within a period of 24 hours.