

Alles gurgelt!

Gargle with Us!



1



Get information and register at allesgurgelt.at. As soon as you have registered, you will receive a personalized barcode.

2



Go to allesgurgelt.at and click on **"take your test now"**. Log in, follow the instructions in the WebApp and take your test.

3



Confirm your identity for the test by holding your passport, ID card or e-card towards the camera.

4



If you agree to a **video recording** of the test procedure, a **medical lab report** will be issued for you for going to a hair salon or other shops/service providers, if your test is negative. If you skip this step, you will receive a test result, but no officially valid medical report.

5



Now perform the **test**. You can gargle or rinse your mouth with the saline solution.

6



After performing the test, put the **sample** back into the LEAD Horizon packaging.

7



Now just hand in the test at **any REWE outlet in Vienna** (BILLA, BILLA Plus, BIPA and Penny as well as all petrol station shops of BP-Merkur Inside, JET-BILLA Stop & Shop, SHELL-BILLA Unterwegs).

8



If you **hand in your sample before 9 a.m.**, you will receive your test result via email within a period of 24 hours.